

18 Litre



B-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° B	Helix - B&D Test	Steam Penetration Test	●	03:30	04:00	~25m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° B	Hollow Wrapped	Solid load, small porous objects, simple hollow load and narrow lumen, multiple casing	●	18:00	15:00	~50m	2.5
134° B	Hollow Wrapped		●	10:00	15:00	~44m	2.5
121° B	Porous	Full porous load	●	18:00	20:00	~57m	1.0
134° B	Porous		●	15:00	20:00	~50m	1.0
134° B	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	15:00	~54m	2.5
134° S	Rapid	Solid load, small porous objects, simple hollow load	●	05:00	10:00	~42m	4.0

REFERRING TO MIDMARK VETASSURE B1B

B-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° B	Helix - B&D Test	Steam Penetration Test	●	03:30	04:00	~35m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° B	Hollow Wrapped	Solid load, small porous objects, simple hollow load and narrow lumen, multiple casing	●	18:00	15:00	~60m	3.5
134° B	Hollow Wrapped		●	10:00	15:00	~54m	3.5
121° B	Porous	Full porous load	●	18:00	20:00	~67m	1.5
134° B	Porous		●	15:00	20:00	~60m	1.5
134° B	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	15:00	~64m	3.5
134° S	Rapid	Solid load, small porous objects, simple hollow load	●	05:00	10:00	~52m	4.5

REFERRING TO MIDMARK VETASSURE B2J

23 Litre

18 Litre



B-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° B	Bowie & Dick Test	Steam Penetration Test	●	03:30	04:00	~25m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° B	Hollow Wrapped	Solid load, small porous objects, simple hollow load and narrow lumen, multiple casing	●	18:00	15:00	~50m	2.5
134° B	Hollow Wrapped		●	05:00	15:00	~39m	2.5
121° B	Solids Wrapped	Solid load, small porous objects, simple hollow load	●	18:00	10:00	~47m	2.5
134° B	Solids Wrapped		●	05:00	10:00	~35m	2.5
134° B	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	15:00	~54m	2.5
121° B	Porous	Full porous load	●	18:00	20:00	~60m	1.0
134° B	Porous		●	05:00	20:00	~50m	1.0
121° B	Rapid	Solid load, small porous objects, simple hollow load	●	18:00	04:00	~41m	4.0
134° B	Rapid		●	05:00	04:00	~30m	4.0
134° B	Hollow Open	Solid load, simple hollow load and narrow lumen	●	05:00	04:00	~36m	4.0

REFERRING TO MIDMARK DENTISURE B18

B-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° B	Helix - B&D Test	Steam Penetration Test	●	03:30	04:00	~35m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° B	Hollow Wrapped	Solid load, small porous objects, simple hollow load and narrow lumen, multiple casing	●	18:00	15:00	~60m	3.5
134° B	Hollow Wrapped		●	05:00	15:00	~49m	3.5
121° S	Solids Wrapped	Solid load, small porous objects, simple hollow load	●	18:00	15:00	~57m	3.5
134° S	Solids Wrapped		●	05:00	15:00	~45m	3.5
134° B	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	15:00	~64m	3.5
121° B	Porous	Full porous load	●	18:00	20:00	~70m	1.5
134° B	Porous		●	05:00	20:00	~60m	1.5
121° S	Rapid	Solid load, small porous objects, simple hollow load	●	18:00	09:00	~51m	4.5
134° S	Rapid		●	05:00	09:00	~40m	4.5
134° B	Hollow Open	Solid load, simple hollow load and narrow lumen	●	05:00	09:00	~46m	4.5

REFERRING TO MIDMARK DENTISURE B33

23 Litre