

18 Litre



S-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° S	Bowie & Dick Test	Steam Penetration Test	●	03:30	04:00	~28m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° S	Hollow Wrapped	Solid load, small porous objects, simple hollow load	●	18:00	15:00	~51m	1.5
134° S	Hollow Wrapped		●	10:00	15:00	~45m	1.5
121° S	Porous	Small sized porous load	●	18:00	20:00	~61m	0.5
134° S	Porous		●	15:00	20:00	~60m	0.5
134° S	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	15:00	~55m	1.5
134° S	Rapid	Solid load, small porous objects, simple hollow load	●	05:00	10:00	~41m	2.5

REFERRING TO MIDMARK VETASSURE S18

S-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° S	Bowie & Dick Test	Steam Penetration Test	●	03:30	04:00	~28m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° S	Hollow Wrapped	Solid load, small porous objects, simple hollow load	●	18:00	15:00	~61m	2.5
134° S	Hollow Wrapped		●	10:00	15:00	~55m	2.5
121° S	Porous	Small sized porous load	●	18:00	20:00	~71m	1.0
134° S	Porous		●	15:00	20:00	~70m	1.0
134° S	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	15:00	~65m	2.5
134° S	Rapid	Solid load, small porous objects, simple hollow load	●	05:00	10:00	~51m	2.5

REFERRING TO MIDMARK VETASSURE S23

23 Litre

# 18 Litre



S-CLASS

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° S	Bowie & Dick Test	Steam Penetration Test	●	03:30	04:00	~28m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° S	Hollow Wrapped	Solid load, small porous objects, simple hollow load	●	18:00	20:00	~56m	1.5
134° S	Hollow Wrapped		●	05:00	20:00	~45m	1.5
121° S	Solids Wrapped	Solid load	●	18:00	15:00	~53m	1.0
134° S	Solids Wrapped		●	05:00	15:00	~42m	1.0
134° S	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	20:00	~60m	1.5
121° S	Porous	Small sized porous loads	●	18:00	25:00	~66m	0.5
134° S	Porous		●	05:00	25:00	~56m	0.5
121° S	Rapid	Solid load	●	18:00	10:00	~47m	2.5
134° S	Rapid		●	05:00	05:00	~36m	2.5
134° S	Hollow Open	Solid load, simple hollow load	●	05:00	10:00	~42m	2.5

REFERRING TO MIDMARK DENTISURE S18

TEMP AND NAME OF CYCLE		STERILIZABLE LOADS <sup>1</sup>	TECHNICAL SPECS OF THE CYCLES <sup>2</sup>				
				STR	DRY	TOT	KG
134° S	Bowie & Dick Test	Steam Penetration Test	●	03:30	04:00	~38m	0
-	Vacuum Test	Leak Detection Test	●	-	-	~15m	0
121° S	Hollow Wrapped	Solid load, small porous objects, simple hollow load	●	18:00	20:00	~66m	2.5
134° S	Hollow Wrapped		●	05:00	20:00	~55m	2.5
121° S	Solids Wrapped	Solid load	●	18:00	15:00	~63m	2.0
134° S	Solids Wrapped		●	05:00	15:00	~52m	2.0
134° S	Prion	Load related to the Creutzfeldt-Jakob disease	●	20:00	20:00	~60m	2.5
121° S	Porous	Small sized porous loads	●	18:00	25:00	~76m	0.5
134° S	Porous		●	05:00	25:00	~66m	0.5
121° S	Rapid	Solid load	●	18:00	10:00	~57m	3.5
134° S	Rapid		●	05:00	05:00	~46m	3.5
134° S	Hollow Open	Solid load, simple hollow load	●	05:00	10:00	~52m	3.5

REFERRING TO MIDMARK DENTISURE S23

S-CLASS

# 23 Litre